

SLY-100 Vision Tester User Manual

Version: 1.3 Revision Date: 2024.02 Product Name: Vision Tester Nome do produto: Testador de vis ão Nombre del producto: Probador de visi ón Ürün Adı: Görme test cihazı Nome prodotto: Tester della vista Produktname: Sehtester Nom du produit : Testeur de vision Име на продукта: Tecrep за зрение Produkto pavadinimas: Foropteris Nazwa produktu: Tester wizji N ázev produktu: Tester zraku Toote nimi: N ägemistestija Produkta nosaukums: Redzes pārbaudītājs Όνομα προϊόντος: Ελεγκτής όρασης

User Manual Download link: www.yeasn.com/en/ Link para download do manual do usu ário: www.yeasn.com/en/ Manual de usuario Enlace de descarga: www.yeasn.com/en/ Kullanım Kılavuzu İndirme linki: www.yeasn.com/en/ Manuale d'uso Link per il download: www.yeasn.com/en/ Link zum Herunterladen des Benutzerhandbuchs: www.yeasn.com/en/ Lien de t d échargement du manuel d'utilisation : www.yeasn.com/en/ Линк за изтегляне на ръководството за потребителя: www.yeasn.com/en/ Naudotojo vadovo atsisiuntimo nuoroda: www.yeasn.com/en/ Link do pobrania instrukcji obsługi: www.yeasn.com/en/ Uživatelská příručka Odkaz ke stažení: www.yeasn.com/en/ Lietotāja rokasgrāmatas lejupielādes saite: www.yeasn.com/en/ Eyyɛupíδio ɣpήστη Σύνδεσμος λήψης: www.yeasn.com/en/

Preface

Thanks for purchasing and using SLY-100 vision tester.

Please read this User Manual carefully before using this device. We sincerely hope that this User Manual will provide you with sufficient information to use the device.

Our pursuit is to provide people with high-quality, complete-function and more personalized devices. Information in promotional materials and packing boxes is subject to changes due to performance improvement without additional notice. Chongqing Yeasn Science - Technology Co., Ltd. reserves the rights to update the devices and materials.

If you have any questions during using, please contact at our service hotline: (86-023) 62797666, we will be very happy to help you.

Your satisfaction, our impetus!

Information of manufacturer

Name: CHONGQING YEASN SCIENCE - TECHNOLOGY CO., LTD Address: 5 DANLONG ROAD, NAN'AN DISTRICT, CHONGQING, CHINA. Tel: 86-23 62797666

Content

1. Introduction
1.1 Uses
1.2 Characteristics
1.3 Main technical indexes 1
1.4 Nameplate and indications
2. Safety Notice
3. Configuration
4. Assembly
4.1 Attaching Instrument to Ophthalmic Stand9
4.2 Attaching Near Point Rod, Near Point Card and Card Holder
4.3 Attaching face shield 10
5. Preventive inspection
6. Operation Procedures 11
6.1 Spherical Lens 11
6.2 Cylinder Lens 11
6.3 Auxiliary Lens 12
6.4 Cross Cylindrical Lens 13
6.5 Rotary Prism
6.6 Corneal Aligning Device
6.7 Near Point Card 19
6.8 Examination Procedures 19
7. Maintenance
7.1 Daily Care
7.2 Checking and Servicing Procedure
8. Before Requesting Service-Troubleshooting Guide
9. Cleaning & Protection
10. Environmental Conditions and Service Life
10.1 Environmental conditions for normal operation
10.2 Environmental conditions for transportation and storage
10.3 Service life
11. Environmental Protection
12. The responsibility of the manufacturer
13. Optional Accessories - Cylinder Lens

1. Introduction

1.1 Uses

This instrument is applicable with stand and projection for precision measurement of visual functions such as myopia, hyperopia, astigmatism, visual acuity balance, phoria, stereoscopic vision and visual acuity amalgamation.

Contraindications: none

Target groups of patients: adults, children

Intended users: optometrists in hospital ophthalmology and optical shops

Specific qualifications of device users and/or other persons: have a qualification certificate for optometry and glasses.

1.2 Characteristics

△ Unique design of butterfly-shape appearance.

△ Capable of checking up all-sided visual functions, accurate and comfortable in measurement.

△ Exquisite manufacturing technique, with comfortable feel.

△ High quality plated-film used in all optic lenses.

△ Technology and design patents

1.3 Main technical indexes

1.3.1 Spherical Lens Range of measurement: $-19.00D \sim +16.75D$
Step length: 0.25D (being 0.12D when 0.12D auxiliary lens is used)
1.3.2 Cylindrical Lens Range of measurement: $0 \sim -6.00D$
(being $0 \sim -8.00$ D when additional lens is used)
Step: 0.25D (being 0.12D when additional lens is used)
1.3.3 Axis of Cylindrical Lens Range of measurement: $0 \sim 180^{\circ}$, Step: 5°
1.3.4 Cross Cylindrical Lens ±0.25D
1.3.5 Rotary PrismRange of measurement: $0 \sim 20\Delta$
Step: 1 Δ
1.3.6 Prism basal angleRange of measurement: $0 \sim 180^{\circ}$, Step: 5°
1.3.7 Pupil DistanceRange: 50mm~75mm
Step: 1mm
1.3.8 Aggregated Adjustment ∞ , 380mm

1.3.9 Adjustment of Forehead Rest	16mm
1.3.10 vertex Distance	13.75mm
1.3.11 Overall Dimensions	335mm (Length)×310mm(Width)×90mm(Height)
1.3.12 Weight	4.5kg

1.4 Nameplate and indications

Name plate and indications are sticked on the instrument to arise end-users' notice.

In case the name plate is not sticked well or the characters become unclear to recognize, please contact authorized distributors.



Key to symbols of the name plate sticked on the instrument:

	Manufacturer	EC REP	Authorized European representative			
~~~	Date of manufacture	REF	Catalogue number			
SN	Serial number	#	Model number			
CE	CE marking	MD	Medical device			
UDI	Unique Device Identifier		Consult instructions for use			

Name plate sticked on the packaging:

VISION TESTER
REF SLY1/X # SLY-100 CE MD CN
DIM. 424mm×414mm×194mm
G.W. 6.5kg
CHONGQING YEASN SCIENCE - TECHNOLOGY CO., LTD. 5 DANLONG ROAD, NANAN DISTRICT, CHONGQING, CHINA
EC REP Shanghai International Holding Corp. GmbH(Europe) Eiffestrasse 80, 20537 Hamburg, Germany

Key to symbols of the name plate sticked on the packaging:

G.W.	Gross Weight	Gross Weight DIM.		
<u>%</u>	Humidity range identification		Atmospheric pressure range identification	
<b>A</b>	Country of manufacture		Temperature limit	
Ţ	Fragile, handle with care	<u>††</u>	This way up	
Ť	Keep dry		Stacking limit by 5	

## 2. Safety Notice

Please read the following precautions carefully to avoid personal injury, device damages or other possible hazards:

• Use the device indoors and keep it clean and dry; do not use it under inflammable, explosive, high temperature and dusty environment.

• Do not use the device near water; also be careful not to make any kinds of liquid drop onto the device. Do not place the device in damp or dusty places, nor place it where humidity and temperature change quickly.

• Ensure that the equipment is installed firmly and reliably before use; if the equipment falls, it may cause personal injury or equipment failure.

• Do not set the instrument with face down or exert pressure onto surface of lens, and do not touch lens by hand.

• The instrument shall not be put in a damp and dusty room.

• All moving parts can be turned in dual-direction. However, care must be taken to do it, and do not turn it beyond the limit position so as to avoid damage to the device.

• The plastic part (Forehead Rest and spirit level, etc.) that can be scrubbed shall be swabbed down using cotton cloth, and do not use cleaning liquid or other chemicals.

• Vision tester belongs to precision instrument, so do not dismantle it at random.

• When taking the vision tester, one should hold mounting handle (Fig.1) at upper part of the instrument or carry left and right ends of the instrument by both hands (Fig.2).



Fig.1

Fig.2

• Notification: Any serious event related to the device to the user and/or patient shall be reported to the manufacturer and competent authority of the Member State where the user and/or patient is located.

# **3.** Configuration


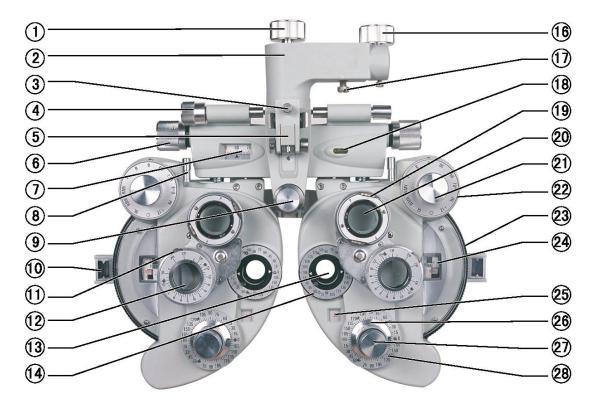


Fig.3

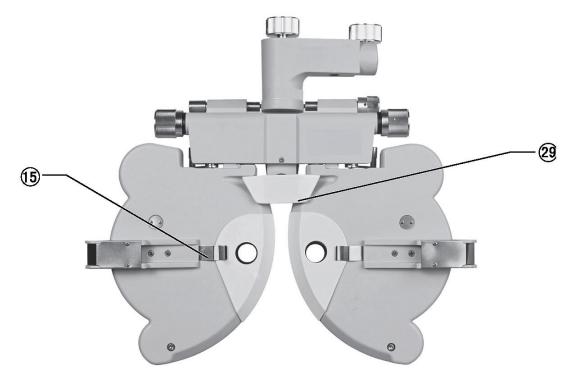



(1) Rotation Adjustment Knob

Used to adjust direction of instrument's main body

2 Mounting Handle

Used to install the instrument onto eye-optometry table

3 Clamping Screw

Used to fix near-point rod

(4) Leveling Adjusting Knob

Used to adjust level position of the instrument

(5) Near Point Rod Holder

Used to attach near-point testing mark rod to hanger frame

6 Pupil Distance Knob

Used to adjust pupil distance

7 Pupil Distance Scale

Used to display pupil distance

8 Vergence Lever

Used to adjust corner of device's left and right disks

9 Forehead Rest Knob

Used to adjust patient's forehead position

(1) Corneal Aligning Aperture

Used to display position of patient's cornea vertex

D Prism Rotation Knob

Used to adjust prism power

Rotary Prism

Used to test phoria or binocular balance

(3) Examination Aperture

Aperture for test, with various lenses set here.

- Cylindrical Lens Axis ScaleUsed to indicate cylindrical lens axis angle
- (5) Face shield chip

Fix face shield

**(b)** Fixing Hand-wheel

Used to fix instrument to ophthalmic stand

Tightening Screw

Used to fix instrument to ophthalmic stand, and stored in accessory box

() Spirit Level

Used to indicate level direction

19 Rotation Knob

Used to adjust astigmatic axis of cross cylindrical lens

20 Cross Cylindrical Lens

Used to precisely check astigmatic power and axis

2 Auxiliary Lens Knob

Used for various visual acuity tests

② Strong Spherical Power Knob

Used to adjust big spherical lens power, step:3.00D

**Q** Weak Spherical Power Dial

Used to adjust small spherical lens power, step:0.25D

② Spherical Power Scale

Used to display spherical lens power

(25) Cylindrical Power Scale

Used to display cylindrical lens power

Q	6	Cylindrical Lens Axis Knob
		Used to adjust cylindrical lens axis
Q	)	Cylindrical Lens Knob
		Used to set cylindrical lens to examination aperture
Q	8	Cylindrical Lens Axis Scale
		Used to display angle of cylindrical lens axis
Q	9	Forehead Rest
		Patient's forehead rests here.
Q	0	Instruction Manual
Q		Near Point Rod
		Card holder is attached to position of near point measuring on this rod.
Q	2	Near Point Card
		Including near point sight mark
Q	3	Dust Cover
		Use dust cover to cover instrument when it is not in use to protect it from dust.
Q	4	Accessories Box
		Used to store the standard accessories
Q	3	Face shield
		Left and right face shield s, one each, are installed on position where instrument
		patient's nose contact.
Q	6	Balloon with Brush
		Used to clean lens
Q	)	Additional Lens (Optional)
		Used to change testing range and precision
-	2	

(3) Internal Screw Angle Wrench

and

Used to install the near point rods

#### (39) Screws

Used to connect two near points rods

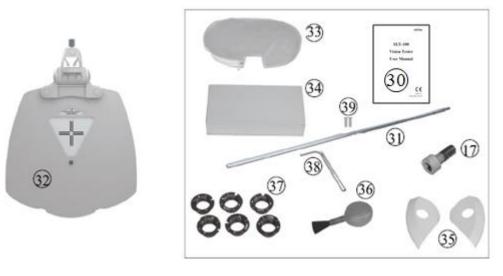




Fig.6

# 4. Assembly

#### 4.1 Attaching Instrument to Ophthalmic Stand

a.When assembly is conducted, first insert the mounting rod extending from the ophthalmic stand to hole of mounting handle ②, and fix it with fixing hand-wheel ①. Then tighten tightening screw ① under mounting handle ②. Tightening screw ① is stored in standard accessories box ③. b.Turn leveling adjusting knob ④ until air bubble is located at middle position of spirit-level bubble ③. Loosen rotation adjustment knob ① to turn the instrument to the required direction. Adjust the device to the right position, then fix the hand-wheel ① again.

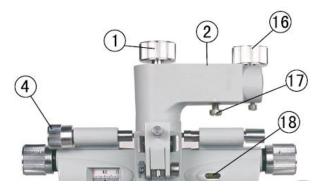


Fig.7


#### *Notice

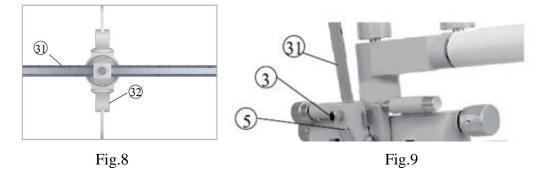Fastening screw ③ (placed in accessories box) can be used to better fasten vision tester when it is not matching with the vision tester holding arm.

#### 4.2 Attaching Near Point Rod, Near Point Card and Card Holder

First, align the connection engravings of the two near point rods, then use 3 internal screw angle wrench to fix the two screws into 3. Secondly, put the near point card 3 into 3 and tighten the top screws of the near point rods (Fig.8). Thirdly, install the 3 onto the 5, tighten the 3.

When 1 is out of use, lift it up(Fig.9).



#### 4.3 Attaching face shield

Attach face shield (5) so that face shield clip (5) catches it. Then align face shield aperture with examination aperture (13) (Fig.10).

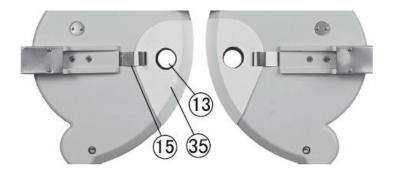


Fig.10

# **5.** Preventive inspection

The equipment management should carry out preventive inspections before use.

The detection window should be clean.

The device is in a horizontal position.

The lenses and accessories are fixed in front of the detection window, and the instrument should be aligned and centered.

Inspection cycle: before use every day.

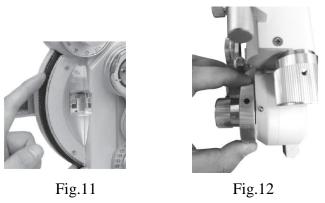
## **6. Operation Procedures**

#### 6.1 Spherical Lens

To show the spherical power only (shortened as "S"), turn auxiliary lens knob (1) to O position, then turn cylindrical lens knob (2) until "00" is shown on cylindrical power scale (2). Then turn weak spherical power dial (2), value S is displayed in spherical power scale (2), within range from -19.00D~+16.75D, increasing or decreasing progressively in 0.25D (Fig.11).

To obtain required diopter setting quickly, use strong spherical power knob (2), then value S increases or decreases progressively in 3.00D diopter steps (Fig.12).

Note: Although several figures will appear on the scale, only three or four digit numbers have meaning. For example, if '075' is shown, it should be read as '0.75D', and if '1150' is shown, it should be read as '11.50D'.



#### 6.2 Cylinder Lens

By turning cylindrical lens knob O, the cylindrical power is shown on cylindrical power scale O, with range from 0.00D to 6.00D, and increases or decreases progressively in 0.25D steps (Fig.13). By turning cylinder lens axis knob O, the axis angle of cylinder lens is shown on cylinder lens axis scale O, with range of 0~180  $\degree$ , step: 5  $\degree$ (Fig.14)


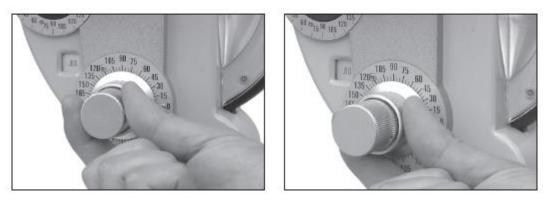




Fig.14

## 6.3 Auxiliary Lens

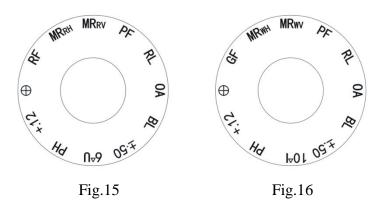Turn auxiliary lens knob (2), the required symbol is to be set at 12 o'clock position. Then Corresponding referenced lens will appear in the examination aperture (13) (Fig.15 and Fig.16).



The meaning of each mark.

OA	Open aperture
BL	Occluder : to block the light path
±.50	Cross cylinder lens, with horizontal plus +axis. Used for presbyopia test
6ΔU	6 diopter base up prism, used for horizontal phoria test
PH	A 1mm diameter pinhole is provided, used to determine reason of poor vision
	(due to refractive abnormity or their reasons)
+.12	+0.12D spherical lens, and spherical power can be set by 0.12D
$\oplus$	Cross piece
RF	Red Filter
MRRH	MR: Maddox rods, RH: Red, Horizontal
MRRV	MR: Maddox rods, RV: Red, Vertical


PF	Polaroid filter, used for polarizing testing of stereoscopic visionand binocular balance of
	stereoscopic vision
RL	Retinoscopic lens; +1.50D spherical lens (67cm)
10ΔI	10 diopter base in prism, used for vertical phoria test
GF	Green color-filter lens
MRWH	MR: Maddox rods, WH: White, Horizontal
MRwv	MR: Maddox rods, WV: White, Vertical
To change	direction of cross cylinder lens and polaroid filter, first remove the retaining ring and
rear cover	glass using screwdriver. Rotate the auxiliary lens knob $\mathfrak{Q}$ until auxiliary lens is
properly in	dexed and in alignment with examination aperture $(3)$ . By slightly turning the auxiliary
lens knob	(1) in both directions, a screw and washer can be seen above and below the lens.
Removing	these two screws, auxiliary lens can be then removed. By reversing above procedure,
reposition of	of lens is possible to ensure it is placed in a proper position (Fig.17).



Fig.17

#### 6.4 Cross Cylindrical Lens

It is used for precise determination of cylinder power and axis. Turn cross cylindrical lens to the front of examination aperture. The letter"P" at its front sustenance face stands for the power, and hand-wheel direction stands for axis. When red dot aligns with "P" it indicates minus -0.25D cylindrical lens. When white dot aligns with "P"it indicates plus +0.25D cylindrical lens.

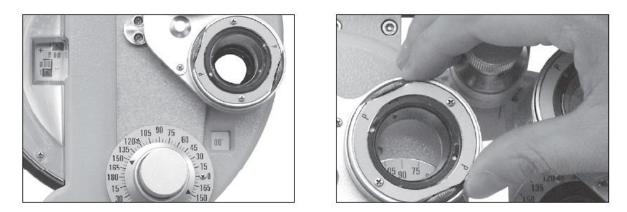





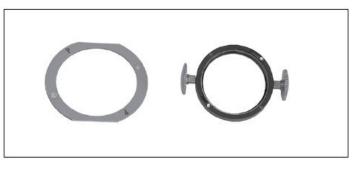


Fig.20

#### 6.5 Rotary Prism

Turn rotary prism (2) by holding its base to set it on the examination aperture. Turn prism rotation knob (1) until the required prism power is set. What the black triangle arrow indicates is the current prism power. For example, prism power indicated in Fig.22 is 0, that in Fig.23 means base in  $3\Delta$  prism power, and that in Fig.24 means base up  $3\Delta$  prism power.

The purpose of below markings:

**•** : Indicate the prism base direction.

When — and 0 are in the horizontal position, the prism base is labeled as vertical direction.
When — and 0 are in the vertical position, the prism base is labeled as horizontal direction.
Current prism base value indicates.


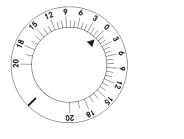







Fig.22

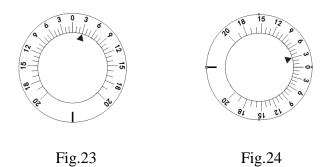

#### 6.6 Corneal Aligning Device

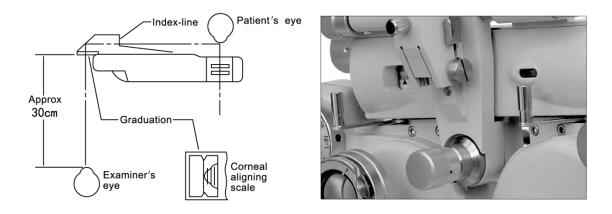

Turn forehead rest knob (9) to adjust position of forehead rest (29). After setting patient's forehead closely on forehead rest (29), look through the corneal aligning aperture (10) from around 30cm away. Look at the apex of the patient's cornea (Fig.25) after the examination aperture's pointer acme of corneal aligning aperture (10) aligns with the longer line on the scale. The longer line in the aperture means the measuring distance is 13.75mm, which is the standard spectacle wearing distance. Three shorter lines are provided by 2mm equal distance away from the longer line. If apex of the cornea of testee is positioned on second shorter line from the longer line, the lens power should be the value measured when spectacle is placed at 17.75mm away from apex of the cornea (standard value 13.75mm + correction value of second shorter line 4mm = 17.75mm). If the actual spectacle wearing distance is different from the standard value (13.75mm), correction should be made according to Table 1 and Table 2.

**Example 1** Assume that data of S +8.00D is obtained when the apex of the cornea positioned at the second shortest line from the longest line, meaning that it is 4mm away from the standard wearing distance. When referring to the correction factor in Table 1, it is known that the applied correction factor is +0.26D for +8.00D diopter and 4mm distance. Therefore, the actual diopter of a patient who wears 13.75 distance standard spectacle is (+8.00D) + (+0.26D)=8.26D. The correction value changes by 0.25 D or 0.12D.

**Example 2** Assume that the apex of cornea is between the second and third shortest lines from the longest line (5mm from the standard line), the obtained data is S-11.50D. It is known when referring the correction factor in Table 2 that for -11.50D and 5mm distance, the correction value should be (0.57+0.68)/2=0.62D. Thus the actual diopter of a patient who wears 13.75 distance standard spectacle is (-11.50) + (+0.62) = -10.88D.

**Example 3** When the cornea apex is on the third shortest line from the longest one, the obtained value is -14.00D: It is known when referring to correction factor in table 2 that for -14.00D and 6mm distance, the correction value should be 1.08D. So the actual diopter of a patient who wears 13.75 distance standard spectacle is (-14.00)+(1.08)=-12.92D.

If more accurate measurement is required, please calculate it according to following formula.

$$D'=D\pm\frac{LD^2}{1000-LD}$$

- D: Measured power
- D': Corrected power
- L: Difference between measured distance and wearing distance(mm)

D	1	2	3	4	5	6	7	8	9	10
+1.00	.001	.002	.003	.004	.005	.006	.007	.008	.009	.01
	.001	.002							.009	.01
+2.00			.01	.02	.02	.02	.03	.03		
+3.00	.009	.02	.03	.04	.05	.06	.06	.07	.08	.09
+4.00	.02	.03	.05	.07	.08	.10	.12	.13	.15	.17
+5.00	.03	.05	.08	.10	.13	.15	.18	.21	.24	.26
+6.00	.04	.07	.11	.15	.19	.22	.26	.30	.34	.38
+7.00	.05	.10	.15	.20	.25	.31	.36	.42	.47	.53
+8.00	.06	.13	.20	.26	.33	.40	.47	.55	.62	.70
+9.00	.08	.16	.25	.34	.42	.51	.61	.70	.79	.89
+10.00	.10	.20	.31	.42	.53	.64	.75	.87	.99	1.11
+11.00	.12	.25	.38	.51	.64	.78	.92	1.06	1.21	1.36
+12.00	.15	.30	.45	.61	.77	.931	.10	1.27	1.45	1.64
+13.00	.17	.35	.53	.71	.90	1.10	1.30	1.51	1.72	1.94
+14.00	.20	.40	.61	.83	1.05	1.28	1.52	1.77	2.02	2.28
+15.00	.23	.46	.71	.96	1.22	1.48	1.76	2.05	2.34	2.65
+16.00	.26	.53	.83	1.09	1.39	1.70	2.02	2.35	2.69	3.05
+17.00	.29	.60	.91	1.24	1.58	1.93	2.30	2.68	3.07	3.48
+18.00	.33	.67	1.03	1.40	1.78	2.18	2.59	3.03	3.48	3.95
+19.00	.37	.75	1.15	1.56	1.99	2.44	2.91	3.41	3.92	4.46
+20.00	.41	.83	1.28	1.74	2.22	2.73	3.26	3.81	4.39	5.00

Correction Table 1 (When Correction Value of Measured Power is in Plus (+) Region)

D L	1	2	3	4	5	6	7	8	9	10
-1.00	.001	.002	.003	.004	.005	.006	.007	.008	.009	.01
-2.00	.004	.008	.01	.02	.02	.02	.03	.03	.04	.04
-3.00	.009	.02	.03	.04	.04	.05	.06	.07	.08	.09
-4.00	.02	.03	.05	.06	.08	.09	.11	.12	.14	.15
-5.00	.02	.05	.07	.10	.12	.15	.17	.19	.22	.24
-6.00	.04	.07	.11	.14	.17	.21	.24	.27	.31	.34
-7.00	.05	.10	.14	.19	.24	.28	.33	.37	.41	.46
-8.00	.06	.13	.19	.25	.31	.37	.42	.48	.54	.59
-9.00	.08	.16	.24	.31	.39	.46	.53	.60	.67	.74
-10.00	.10	.20	.29	.38	.48	.57	.65	.74	.83	.91
-11.00	.12	.24	.35	.46	.57	.68	.79	.89	.99	1.09
-12.00	.14	.28	.42	.55	.68	.81	.93	1.05	1.17	1.29
-13.00	.17	.33	.49	.64	.79	.94	1.08	1.22	1.36	1.50
-14.00	.19	.38	.56	.74	.92	1.08	1.25	1.41	1.57	1.72
-15.00	.22	.44	.65	.85	1.05	1.24	1.43	1.61	1.78	1.96
-16.00	.25	.50	.73	.96	1.19	1.40	1.61	1.82	2.01	2.21
-17.00	.28	.56	.82	1.08	1.33	1.57	1.81	2.04	2.26	2.47
-18.00	.32	.63	.92	1.21	1.49	1.75	2.01	2.27	2.51	2.75
-19.00	.35	.70	1.02	1.34	1.65	1.94	2.23	2.51	2.77	3.03
-20.00	.39	.77	1.13	1.48	1.82	2.14	2.46	2.76	3.05	3.33

**Correction Table 2** (When Correction Value of Measured Power is in Minus (-) Region)

#### 6.7 Near Point Card

If the lens is multifocal, it is required to measure lens diopter at near distance. Then near point rod (3), near point card (3) can be used. Lower near point rod (3), keeping rod horizontal is the correct setting for measurement (Fig.27)

Near point distance from 15cm to 70cm (i.e. about 6 inches to 28 inches), and lens diopter from +8D to +1.5D are provided. The value indicated at tail of card holder #3 is just the value to the card from apex of the cornea (Fig.28). Select the required sight mark on the near point card. Turn the rotating portion along the card center by the finger until the required value appears in the sight window.

Attention: The recommended distance for close optometry is 40 cm, and the size of the visual markings is designed according to the distance of 40 cm.



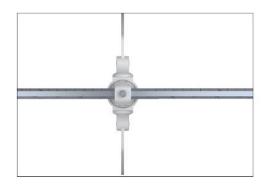


Fig.27

Fig.28



Fig.29

Then turn vergence lever (8) inward to move the instrument so that main axis of lens faces to 380mm. Now near point testing can be carried out (Fig.29).

#### **6.8 Examination Procedures**

Following is an example of examination. Before examination, patient's visual acuity should be

determined.

Example: Testee, 35 years old, who wears spectacles.

First, use lensmeter to measure the spectacles he is wearing, with following results:

PD 63mm R -1.00DS/-0.50DC 90° L -1.25DS/-0.50DC 180°

Examining results show pupil distance of Testee is 63mm; spherical power of his right eye is -1.00D, with astigmatic power of -0.50D, and axis  $90^{\circ}$ ; spherical power of his left eye is -1.25D, with astigmatic power of -0.50D, and axis  $180^{\circ}$ .

With these spectacles worn in examination, visual acuity of Testee's left and right eyes is all 0.7(20/30). Then use a comprehensive optometry meter to precisely measure diopter power of Testee's left and right eyes at present.

#### **6.8.1 Installing Instrument**

(1)Attach the near point rod 1 downwards to near point rod holder 5 (Fig.9).

(2)Set the spherical lens power (value S) and cylinder lens power (value C) to zero.

(3)Before examination, first set pupil distance. Turn pupil distance knob (6), so that pupil distance

of Testee is shown in pupil distance scale (7).

(4) Move the instrument so that the instrument's side shown in Fig.4 is facing to Testee. Now place forehead of Testee on the forehead rest 29.

(5)Turn leveling adjusting knob ④ while observing air bubble until the air bubble moves to middle of the water bubble.

(6) Determine the distance between vertex of cornea and the instrument.

(7) To measure right eye first, turn auxiliary lens knob to set O for right eye, and OC for left eye.

#### 6.8.2 Examination Using "Fogging Method"

(1)Add 3.00D to the estimated S value for right eye. Then power of his spectacle is -1.00D, namely, (-1.00)+(+3.00)= +2.00D.

(2)In this condition, Testee is unable to clearly see the projected chart. Gradually add minus power. In

example of Testee , reduce S value gradually by turning weak spherical power dial  $(3:2.00 \rightarrow 1.75 \rightarrow 1.5 \rightarrow 0.5 \text{ until it shows -1.00 D}.$ 



Fig.30

Fig.31

(3)Project the astigmatic chart while asking Testee if he can see it. If Testee says he can see it as shown in Fig.30, turn the cylindrical lens axis knob 0 to 90° from the darkest line he has seen (see Fig.31). If Testee says all lines are equally bright, it means no astigmatism is existent. Then procedures (4) and (5) in 6.8.2 and procedure 6.8.3 are not required.

(4) Turn cylindrical lens knob O to change C value,  $.00 \rightarrow .25 \rightarrow .50$  so every line is seen equally. When it is turned to -0.50, the chart is as shown as Fig.32.

(5) Change S by 0.25D steps by turning weak spherical power dial 3 so that the visual acuity becomes from 1.2 to 1.5. Record the changed value of the visual acuity.

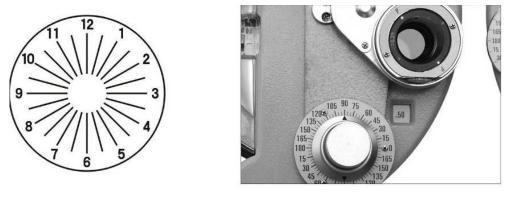


Fig.32

Fig.33

For myopia, spectacles with least power should be selected, and for presbyopia, spectacles with largest power should be selected. To correct Testee's vision to that of 1.5, his spectacle power may be -1.75, -2.00 or -2.25, and then -1.75 should be selected. Now examination is almost completed, however, more precise measurement is required.

#### 6.8.3 Precise Refining Cylinder Axis and Power

(1) Set cross cylindrical lens ② in front of Testee's right eye and, turning rotation knob ③ axially, to align it with axial direction of cylindrical lens (see Fig.33).

(2) Project the cross cylinder dot chart as shown in Fig.34. Turn rotation knob 9 with finger to rotate cross cylindrical lens 9. Then ask Testee to compare the two images he sees before and after turning the cross cylindrical lens. Stop at the better side. For example, if what Testee sees is clearest as shown in Fig.35 of cross cylindrical lens, turn cylindrical lens axis knob 9 to move axis of cross cylindrical lens by 5° in direction of red dot, so that position of cylindrical lens axis scale 9 is positioned at 95°.

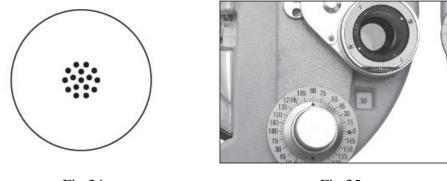


Fig.34

Fig.35

(3) Turn the lens again to make a comparison. If what Testee sees is the most clearest as shown in Fig.37, move cross cylindrical lens axially towards red dot by 5°, enabling it to become 100°.
(4)Turn the lens again. If Testee cannot report any difference, precise examining cylinder axis is completed (with astigmatic axis of 100°).

(5) Now to conduct precise measurement of cylinder power (C), and turn letter P to original axis (see Fig.37).

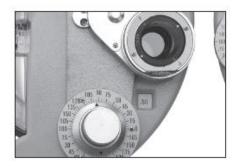


Fig.36


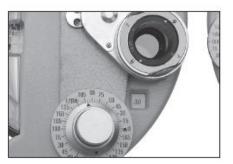



Fig.37

(6) Use cross cylinder dot chart shown in Fig.34 with same procedure as described in (2). Now ask Testee to compare the charts he sees. The result is shown as Fig.38. If Testee sees the clearest chart when red dot matches with letter P (as shown in Fig.38), it means Testee's diopter has increased by 0.25D (now Testee's diopter power is 0.75D).

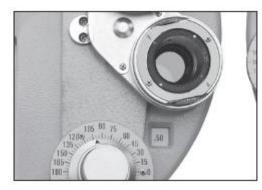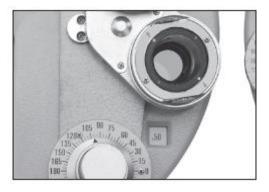(7) Turn the lens again to make a comparison. If the chart as shown in Fig.39 is the clearest, the diopter power should be decreased by 0.25D because white dot is positioned at P. If the red dot is positioned at P, it means diopter power is increased by 0.25D, thus totally 0.5D is added.









(8) Turn the lens again to verify the finding. If Testee reports that the chart in the setting of Fig.39 is clearest, the correct modified power should be between 0.25D and 0.5D. Hence the accurate power should be -0.62D.

#### 6.8.4 Precise Refining Spherical Power (Red-Green Test)

(1) Use red and green chart to determine precise spherical lens value (see Fig.40). Ask the patient which one is seen clearest, red or green chart. If the green one is seen better, it indicates myopia is increased (hyperopia decreased). The reduce spherical lens value by 0.25D.  $-1.75 \rightarrow -1.50$ .

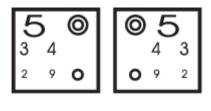


Fig.40

(2) Ask Testee again to affirm which chart is seen clearer, the clearer red stands for decreased myopia (increased hyperopia). Testee's power is 1.62D. Generally, the weak spherical power dial is used to adjust myopia (and strong spherical power dial is used to adjust hyperopia).

(3) Now right eye examination is completed, with lens power result as follows:

Spherical power 1.50 Cylinder power 0.50 and Axis  $100^{\circ}$ 

R -1.50DS/-0.50DC 100°

Then examine the left eye. Turn auxiliary lens knob (2), to set O for left eye, and OC for right eye.


Then use same measuring method to measure left eye.

Testee 's left eye is measured as: L -2.00DS/-0.50DC  $170^{\circ}$ 

#### 6.8.5 Binocular Balance Test

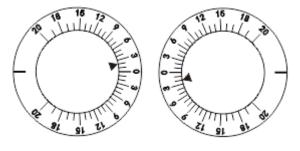(1) Rotary Prism Method

a. Tests are performed for left and right eyes independently, in which, binocular prism shall be used for both eyes. On the whole, these tests are referred to as the binocular balance test. Set both eyes to O. Use the chart shown in Fig.34 and set prisms as  $2\Delta U$  (right eye), and  $2\Delta D$  (left eye) (see Fig.41)





b. Now Testee sees two images of chart, one at upper side and one at lower side. When asked which image is seen clearest, Testee replies the upper one is clearest. Then add +0.25D to spherical lens value of right eye. When the image at lower side is seen clearest, add +0.25D to spherical lens value of left eye, namely, (-2.00)+(+0.25)=-1.75D.

c. Ask Testee again to affirm which one is clearest. When both become similar, it means balance test is completed.

d. Remove the rotary prism. Add spherical lens power of + 1.00D to both eyes. So, Testee's visual acuity should be:

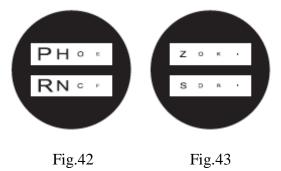
- R -0.50DS /-0.50DC A 100  $^\circ$
- L -0.75DS /-0.50DC A 170  $^\circ$

e. Now add minimum power of 0.25D to binocular spherical lens value . Gradually change the spherical lens value until he can see 1.2 or 1.5(20/15) visual mark clearly . He desires to see 1.5(20/15) clearly, then change spherical lens value as follows:

R -1.50DS /-0.50DC A 100  $^{\circ}$ 

- L -1.75DS /-0.50DC A 170 °
- (2) Polarizing filter method

a. Turn auxiliary lens knob (1) to P (both eyes). Project the polarized binocular balance test chart.



b. Now Testee sees two images, one at upper side and one at lower side. When asked which image is seen clearest, Testee replies the upper one is clearer, and he can see upper row of the chart by his right eye, and the lower row by his left eye. If both rows can be seen with equal clarity, it means the balance is good. When both rows are not seen with equal clarity, add +0.25D spherical lens value to one eye with better clarity until both columns are seen with equal clarity.

c. Turn auxiliary lens knob 0 to O (both eyes). Add +1.00D to spherical lens value of both eyes.

d. Gradually reduce spherical lens value by precision of minimum 0.25D until visual acuity for both eyes becomes 1.2 or 1.5.


#### 6.8.6 Measuring Phoria at Far Point

(1) Maddox rod and rotary prism method

a. First conduct horizontal phoria measurement. Proceed according to (1) rotary prism method described in 6.8.5 Binocular Balance Test. Turn auxiliary lens rotation knob (1), and set right eye to MRRH (Fig.44). Turn prism rotation knob (1) with its setting 0 on the triangle symbol facing the left eye. Light a small fixation light at the position where the chart is projected. Now Testee's right eye can see a red vertical line (see Fig.45 a), and his left eye can see a light spot (see Fig.45 b). They are probably (a) or (b) of Fig.46. The light spot will also move when prism rotation knob (1) is turned. Then ask the patient to tell when he sees the image shown in Fig.46 b. The test result is shown in Fig.47. The prism rotation scale is shown as 2. The result of  $2\Delta I$  (base inward) stands for 2

#### $\Delta$ inclination outward.

b. Then measure vertical phoria. As shown in Fig.48, turn auxiliary lens knob (1) and set MRRV for the right eye. Turn rotation prism lens (1) to set the left eye at horizontal position. Now Testee can see red horizontal line with the right eye, and the light spot with the left eye. Then using same procedure of a, ask Testee when he can see red line and light spot meet while turning prism lens rotation knob (1). When it is shown as Fig.49, Testee reports they meet, it is 0.5, below 0, indicating left eye is  $0.5\Delta D$ , called  $0.5\Delta upward$  heterophoria.

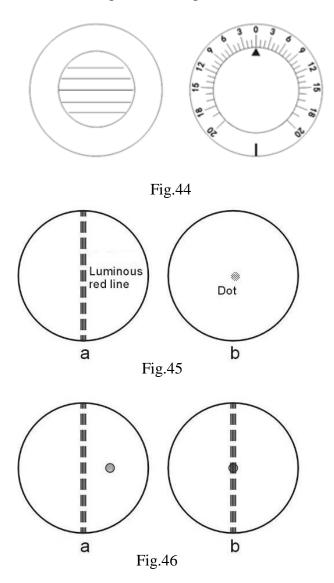

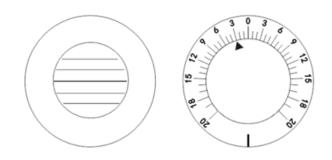


Fig.47

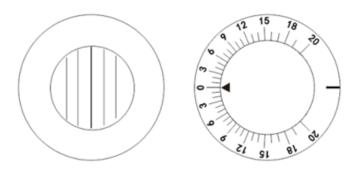


Fig.48


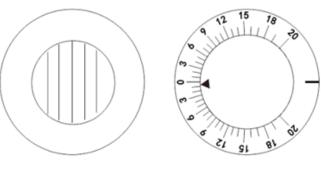


Fig.49

(2) Polarizing Filter Method

a. Turn auxiliary lens knob (2) to P and project the polarizing chart (Fig.50).

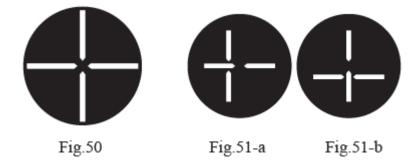b. Unless the patient has phoria, four lines seen to the patient will be shown as Fig.50. If the patient has phoria, these four lines will not be in alignment.



c. When the vertical lines are seen disposed as shown in Fig.51-a, turn rotation prism (2) of the left eye with 0 scale upward. Then turn prism rotation knob (1) slowly so that the image is shown as Fig.50 (horizontal phoria).

d. When horizontal lines are seen disposed as shown in Fig.51-b, adjust 0 scale to horizontal position, and then turn prism rotation knob so that the image is as shown in Fig.50 (vertical phoria).

e. When both vertical and horizontal lines are disposed to have phoria, as shown in Fig.51-c, adjust rotary prism 0 to make scale 0 vertical so that the vertical line is in the middle of horizontal line, as shown in Fig.51-b (horizontal phoria). Afterwards, adjust scale 0 to be horizontal. Turn prism rotation knob 0 so that horizontal lines are in the middle of vertical line, as shown in Fig.51-a (vertical phoria).



#### **6.8.7 Arranging Results**

Now, examination of Testee is completed. If the results show Testee has severe phoria, the spectacles should be adjusted. If not, the prescription would be:

PD 63mm

- R -1.5DS/-0.5DC 100°
- L -1.75DS/-0.5DC 170°

#### 6.8.8 Presbyopia Test

This test is provided to those who are more than 45 years old.

a. First, affirm distance of measurement and put it in the examination aperture. Attach near point rod (1) and near point rod holder (5) to the instrument, then fix them firmly using clamping screw (3).

b. Turn auxiliary lens knob (1) to  $\pm .50D$  (both eyes).

c. Use near point card O as near point examination of the patient. Ask the patient how about the vertical line and horizontal line he sees. If presbyopia is seen, the horizontal line will be seen clearly, with vertical line being dull (if both lines are seen equally, presbyopia spectacles are unnecessary).

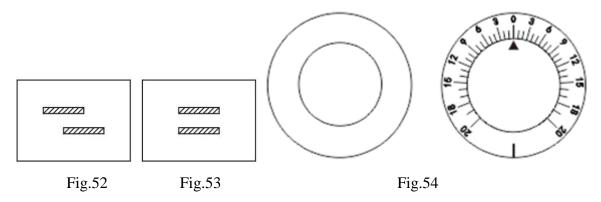
d. Add 0.25 to both eyes' S simultaneously until horizontal line and vertical line are equally discernible.

e. Change  $\pm 50$  of both eyes to O. Turn the near distance card to show small letters. Then ask the patient if letters are clear. A proper adjustment is required for S value. The measurement is complete. Record the results.

#### 6.8.9 Phoria at Near Distance


#### (1) Horizontal Phoria

If the patient has no presbyopia, set the results of phoria tested at far point in aperture. If the patient has presbyopia, put the results on the near point test. Set the near point card at 40cm, and turn auxiliary lens knob (2) to set the right eye at  $6\Delta U$  so that letter rows are fully separated. If the patient has horizontal phoria, it will be shown as Fig.52. Turn rotary prism (12) to the other eye, with 0 scale upward.(see Fig.54) Turn prism rotation knob (1) so that there is no difference between the left and right eyes and, at this time, scale of the rotary prism indicates prism power(see fig.53).



#### (2) Vertical Phoria

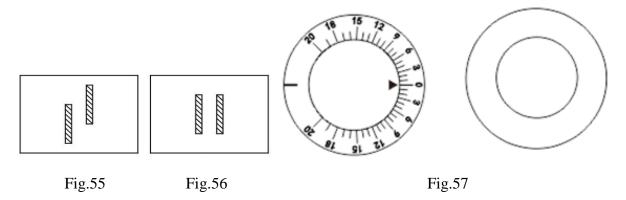Turn auxiliary lens knob (2) to set the left eye at  $10\Delta I$  so that letter columns are completely separated. If the patient has vertical phoria, it will be shown as Fig.55. Then turn the rotary prism to the other eye, with scale of 0 horizontal (as shown in Fig.57). Turn prism rotation knob (1) so that there is no difference between the upper and the lower (see Fig.56). Then scale of rotary prism indicates vertical phoria power.



#### 6.8.10 Other Measurements

(1) Vergence (eyeball movement in different direction)

Set rotary prism (2) in front of both eyes, and place 0 setting in uppermost position. To measure adduction of eyeball at far point, turn the prism outward for both eyes simultaneously. When the chart is seen as two images in the vertical direction (the point where double vision first occurs), the reading at this time indicates adduction power. The rotary prism can be used to measure maximum  $40 \Delta \text{ only}$  (about  $22^{\circ}$ ). For abduction measurement, turn the prism of both eyes inwards simultaneously. When the object is seen as a double image, record the readings. Maximum range of measurement is  $40\Delta$ . If  $10\Delta$ BI is used on auxiliary lens disk, maximum test value is  $50\Delta$ . Adduction

and abduction at near point can be measured when the near point card is fixed to near point rod (3). The method for other measurements is identical.

(2) Vertical Abduction

Set rotary prism (2) in front of both eyes, and place 0 setting in horizontal position. Use the horizontal letters in the visual acuity chart for far point (5m) test, and use near point card to conduct near point test. Turn prism rotation knob (1) and, when horizontal letters are seen as a double image, record the reading, which is patient's vertical abduction power.

#### 6.8.11 Transposition of Prescriptions

In Comprehensive Optometry Device, myopia astigmatic method is used to carry out fogging measurement. However, when hyperopia astigmatism is required sometimes, please use correction results in following formula.

XDS/YDC AZ° $\rightarrow$ (X+Y)DS/(-Y)DC (Z $\pm$ 90) °

S: Add the cylinder lens power to the spherical lens power

C: Convert the index (+-) of cylinder lens power

A: Add 90 ° when Z is less than 90 °, and deduct 90 ° when Z is larger than 90 °.

Example 1:

For +4.00DS/-1.50DC  $\times$ 155 °, changed to:

S: (+4.00)+(-1.50)=+2.50

C: - 
$$(-1.50) = +1.50$$

A: 155 °-90 °= 65 °

So the result is

+2.50DS/+1.50DC  $\times\!\!65~^\circ$ 

Example 2:

For +1.5DS/+0.75DC ×75 °

S: (+1.5)+(+0.75)=+2.25

C: -(+0.75)= -0.75

So the result is:

+2.25DS/-0.75DC ×l65  $^\circ$ 

#### 7. Maintenance

#### 7.1 Daily Care

(1) Use dust cover 3 to protect the instrument from dust when it is not in use.

(2) For long-term storage, keep the instrument in a dry place free of dust.

(3) When lens becomes dirty, use lens cleaning cloth moistened with a little absolute alcohol to wipe it.

(4) Before operation. Clean Forehead Rest ② and nosepiece with medical cotton watted with absolute alcohol.

#### 7.2 Checking and Servicing Procedure

In normal use, no special checking or servicing is necessary. However, when it is used at extremely low temperature, the turning knobs or dials will become heavier than usual because of lubricant used inside, instead of any mechanical reason. When temperatures return to normal, all will be normal.

If there is any failure, do not disassemble and repair it by yourself, please contact your local dealer or manufacturer.

The company promises to provide the user with the necessary parts list and other related materials for repairing the equipment according to the user's needs. Repairable and replaceable parts, such as forehead support, can only be used by our company; use of unapproved parts may reduce the minimum safety of the equipment.

# 8. Before Requesting Service-Troubleshooting Guide

If any problem occurs, first check the following items, and follow the suggested instructions. When the trouble cannot be eliminated, please contact us.

(1) The required lens cannot be set at examination aperture

Is knob turned to correct position?

Is any other lens attached to the patient's examination aperture?

(2) When vergence lever (8) is adjusted, does any action of corresponding vergence occur?

Whether the PD is less than 55mm? When the PD is less than 55mm, steering adjustment can't be

processed.

# 9. Cleaning & Protection

Note: When cleaning, do not wipe with any corrosive detergent to avoid damage to the surface.

Note: Do not wipe with hard cloth, hard paper, etc. Otherwise, the detection window glass may be scratched.

Note: Wipe gently when cleaning the detection window. Otherwise, excessive force may scratch the detection window.

(1) When the instrument is not in use, use a dust cover to prevent dust.

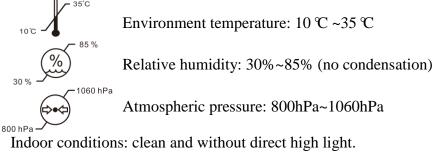
(2) For long-term storage, the instrument should be placed in a dry and dust-free place.

(3) When the lens is dirty, wipe it with a trial lens cloth and a small amount of absolute alcohol.


(4) Before optometry, wipe the forehead and nose pads with medical cotton and a small amount of absolute alcohol.

# **10. Environmental Conditions and Service Life**

#### 10.1 Environmental conditions for normal operation



#### 10.2 Environmental conditions for transportation and storage

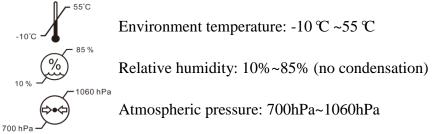

#### **10.3 Service life**

The service life of the device is 8 years from first-time use with proper maintenance and care.

# **11. Environmental Protection**

In order to protect the environment, please pack the equipment and send it back to our company when the equipment life expires, or dispose of it in accordance with local regulations in force.

# 12. The responsibility of the manufacturer

The company is only responsible for the impact on the safety, reliability and performance of the equipment under the following conditions:

- Assembly, addition, adjustment, modification or maintenance are all carried out by personnel approved by the company;

- This equipment is used in accordance with the requirements of the user manual.

# 13. Optional Accessories - Cylinder Lens

Three types of spare lens are optional: -2.00CYL,-0.12CYL and 00CYL.